


NeoPulse™

Disposable Pulse Oximeter Wrap

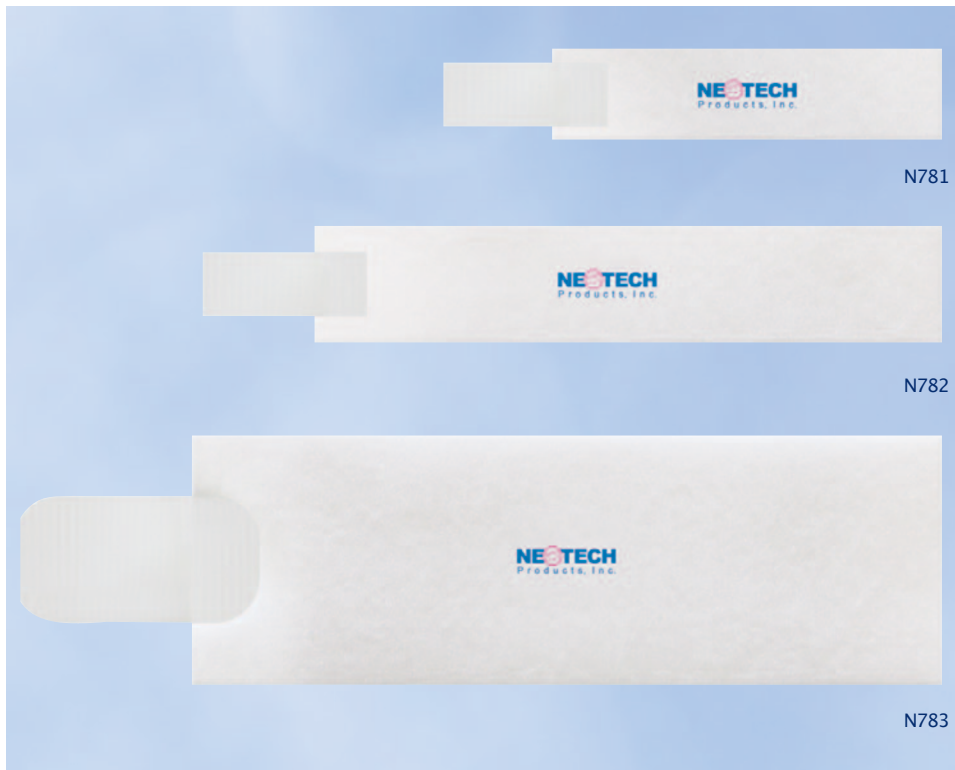

FEATURES AND BENEFITS

- Soft, breathable, skin friendly NeoFoam® wicks perspiration
- Lightweight, form fitting and adjustable
- Designed for comfort and stability
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

Cat No.	Size	Dimensions	Qty/Unit
N781	Small	3/4" x 3"	100/box
N782	Large	1" x 5"	100/box
N783	X-Large	2" x 6"	100/box



NeoPulse™

Disposable Pulse Oximeter Wrap

INDICATIONS FOR USE

The NeoPulse is intended to hold a pulse oximeter sensor in place. It is intended for use on pediatric (neonates, infants and children) patients.

DIRECTIONS FOR USE

Step 1

Place pulse oximeter sensor per hospital protocol.

Step 2

Wrap NeoPulse around foot and pulse oximeter sensor and gently fasten hook tab to wrap.

Note: May be trimmed for proper fit. Ensure that wrap is not too tight. Hook tab should completely overlap wrap.

Step 3

Check foot after wrapping to ensure adequate circulation.

Step 4

Change the NeoPulse every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if visibly worn or soiled.



Pat Coakley Medical Ltd

Units E3 E4 Network Enterprise Park
Kilcoole A63 H244 Co Wicklow
sales@patcoakleymedical.ie

Tel: +353 1 2018070
www.patcoakleymedical.ie



GENERAL INFORMATION

Product Materials

NeoFoam, hook fabric
Made with natural rubber latex: No
Made with plasticizer DEHP: No
Consult MSDS for complete material information.

Size

N781 Small: 75 mm x 20 mm
N782 Large: 125 mm x 25 mm
N783 XL: 150 mm x 50 mm

Packaging

Individually packaged: Yes
Package type: Polyethylene bag
Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information or to download the Material Safety Data Sheet (MSDS).

Polyethylene bags are biodegradable. Boxes are printed with environmentally friendly soy ink on recyclable material.

CAUTIONS & WARNINGS

Discontinue immediately if skin irritation occurs.

Single patient use only.